



Lincoln Radiology Imaging & Interventional  
PACS Viewer Install notes  
7121 Stephanie Lane Suite 100  
Lincoln, NE 68516  
(402) 420-3500  
Fax (402) 423-4100

### **Lincoln Radiology PACS Viewer Install Notes**

<https://images.lincolnradiology.com/amicas-reach/>

2.)	Add LincolnRadiology.com to the IE Security Settings.....	4
3.)	Removing “reuse windows” for launching shortcuts.....	10
4.)	Java Installation (Java 1.5.0.19).....	11
5.)	Clear Internet Cookies, Files and History.....	18
5.)	Adding a Shortcut .....	21
	Miscellaneous Notes: .....	24

**If you have any questions are in need of any assistance,**

please contact:

Tony Curtis  
(402) 420-3536  
[curtis@lincolnradiology.com](mailto:curtis@lincolnradiology.com)


1.) Add LincolnRadiology.com to any pop up Blockers

**This Outlines Adding lincolnradiology.com to the Internet Explorer (Version 6.x) Pop – Up blockers.**


If you are using pop up blockers make sure to allow pop-ups from lincolnradiology.com.

If you are using Microsoft’s Internet Explorer pop-up blocker, you can add this by doing:

In Internet Explorer Menu Tools, Pop-up Blocker, Pop-Up Blocker Settings

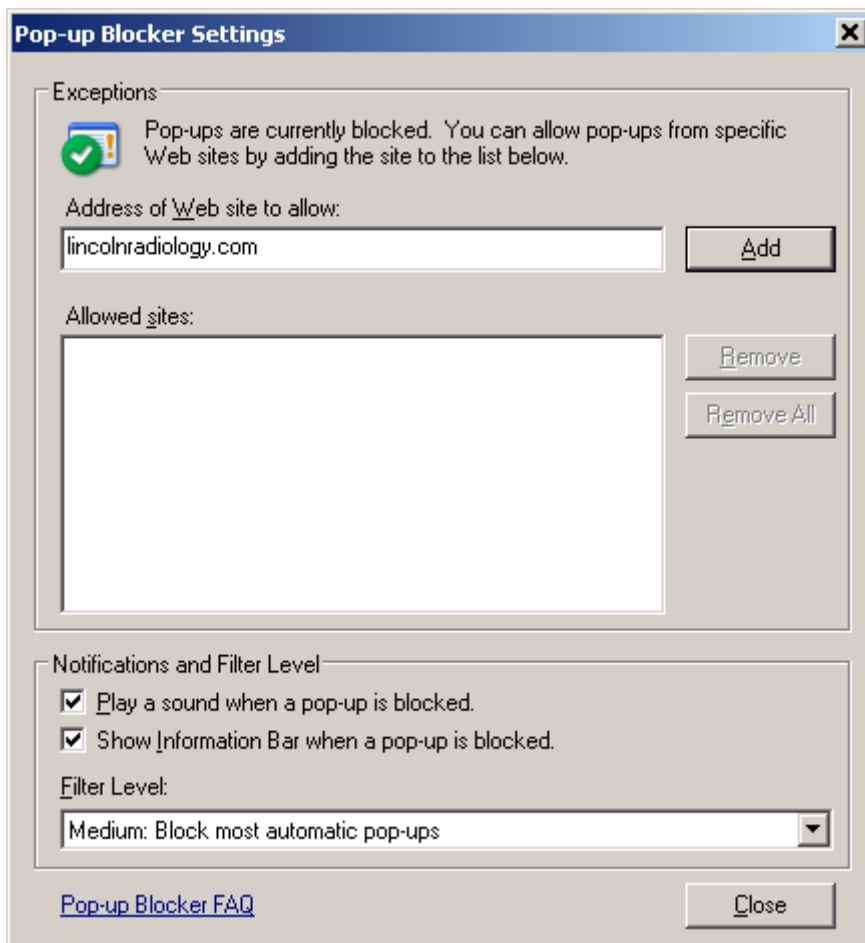

Type in lincolnradiology.com and select “Add”

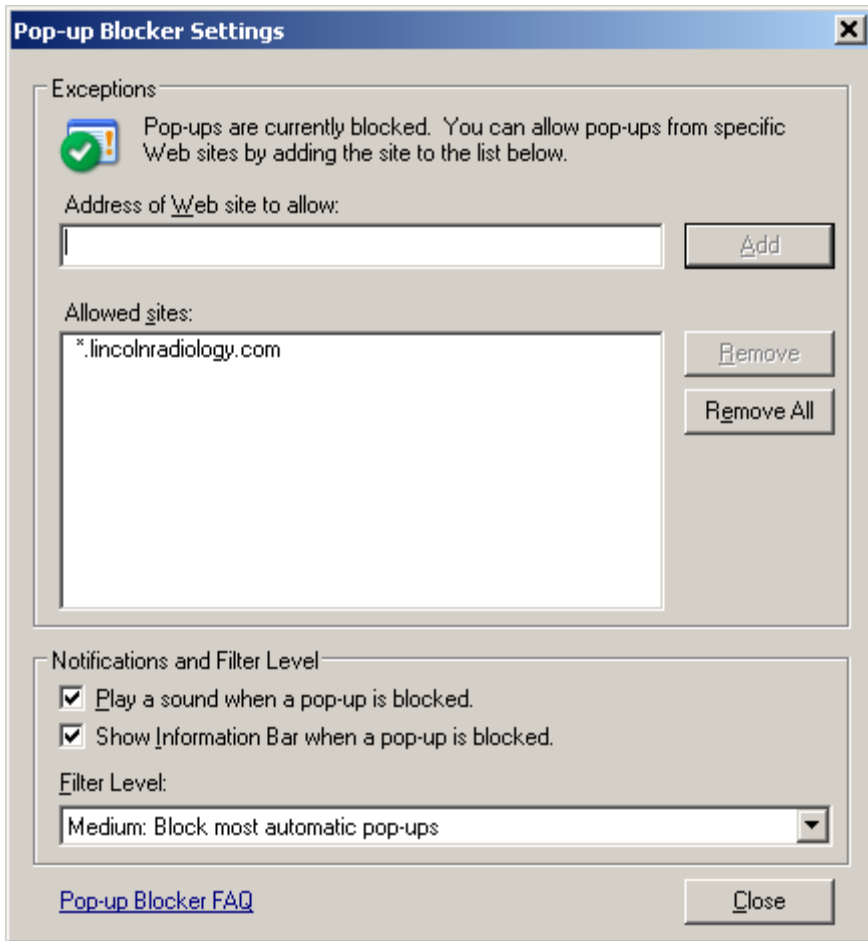


After selecting add, you now can select “Close”.

Note if you wish to add more specific sites add the following two sites.  
<https://images.lincolnradiology.com>  
<https://pacs.lincolnradiology.com>


## 2.) Add LincolnRadiology.com to the IE Security Settings

In Internet Explorer we need to add lincolnradiology.com as a trusted site.  
{**Vista-For installation**, make sure to run Internet Explorer as Administrator-  
Start, Programs, right click on Internet Explorer, run as Administrator}



From Internet Explorer, select Tools, Internet Options

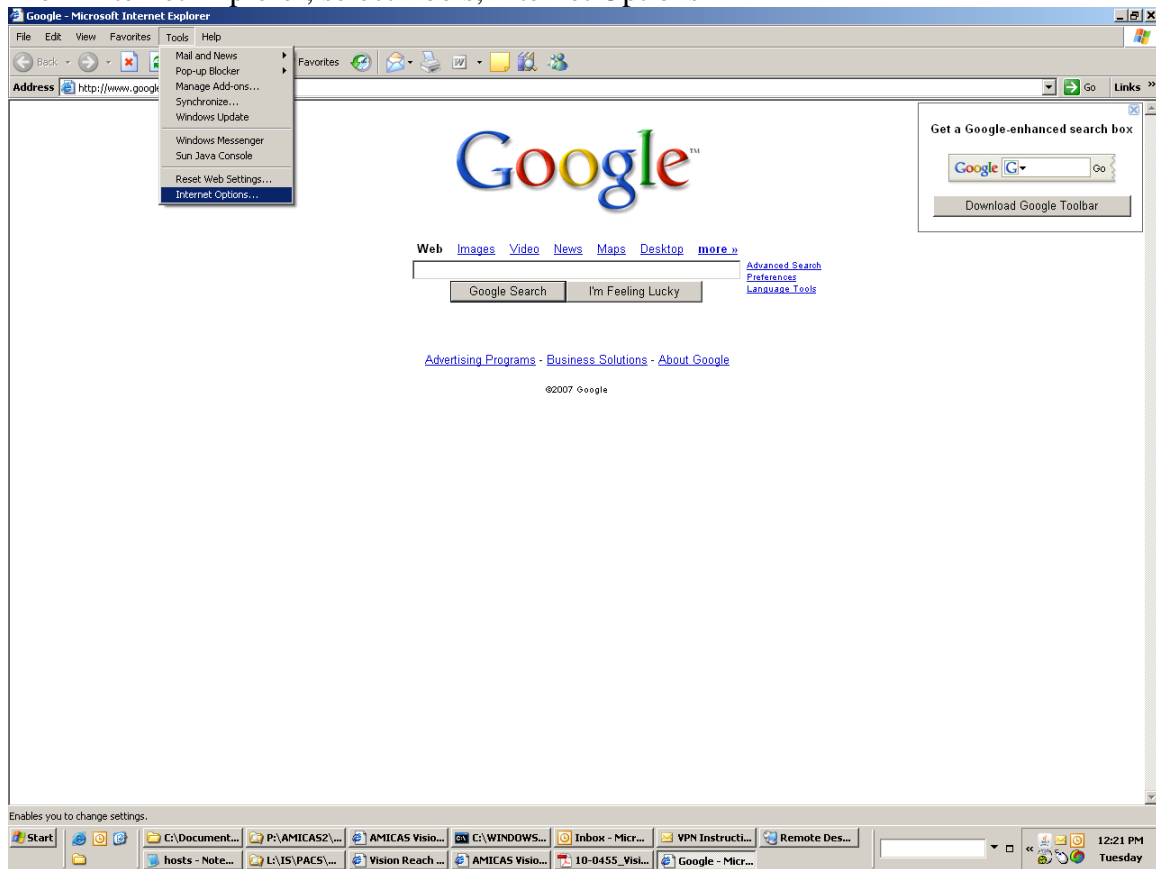


Select the Security tab, and make sure “Trusted Sites” are highlighted, and then select Sites.

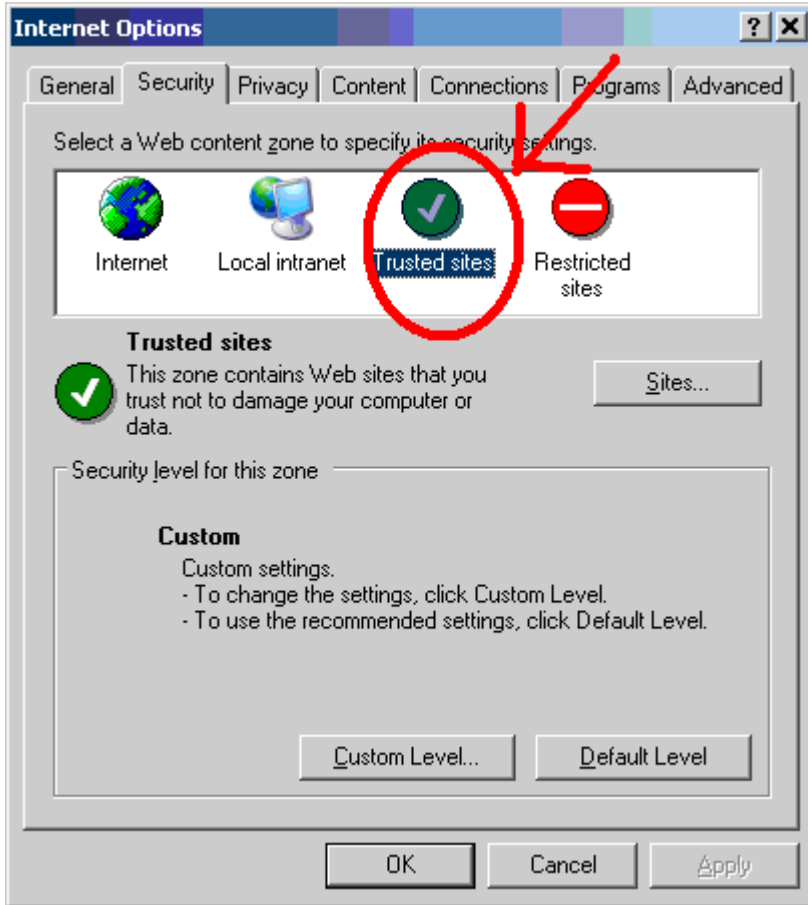

Enter <https://lincolnradiology.com> and then select Add.

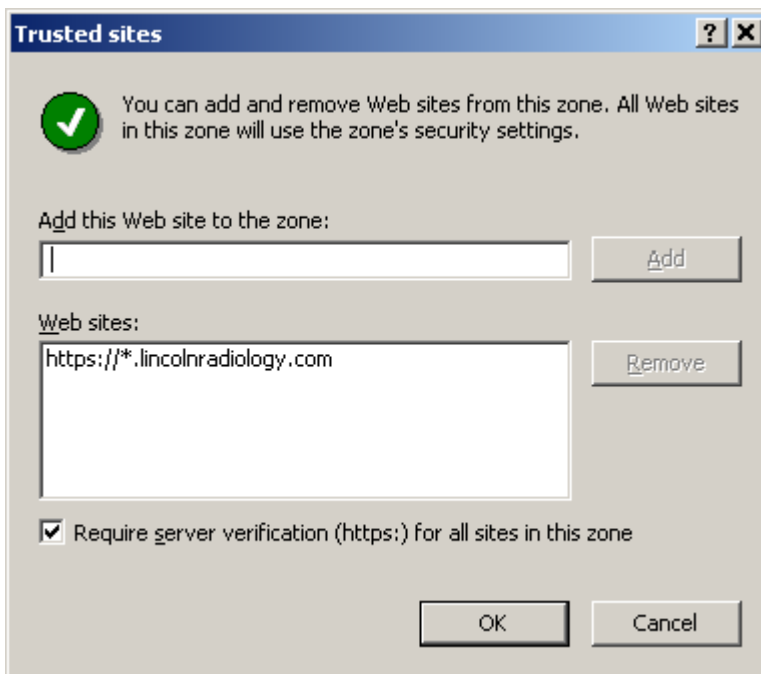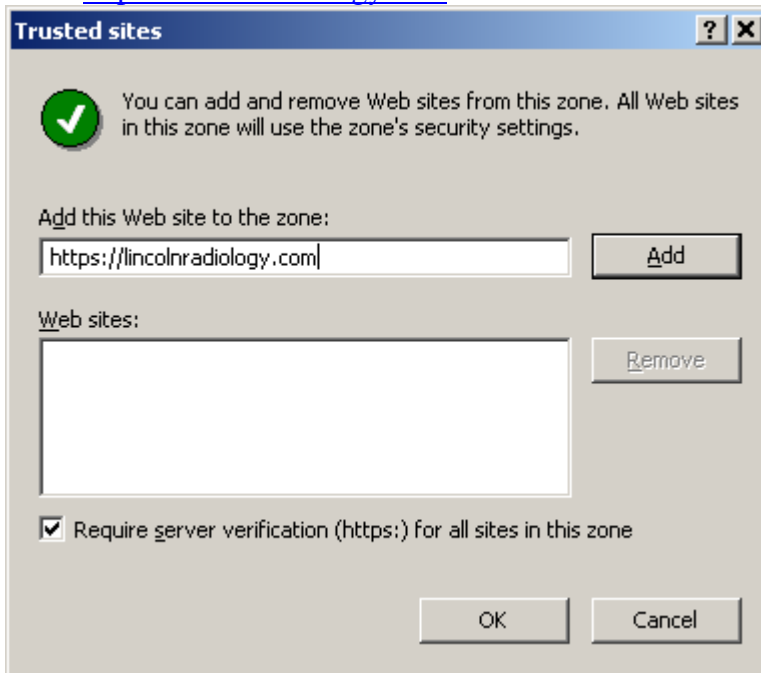


Now you can select “OK”

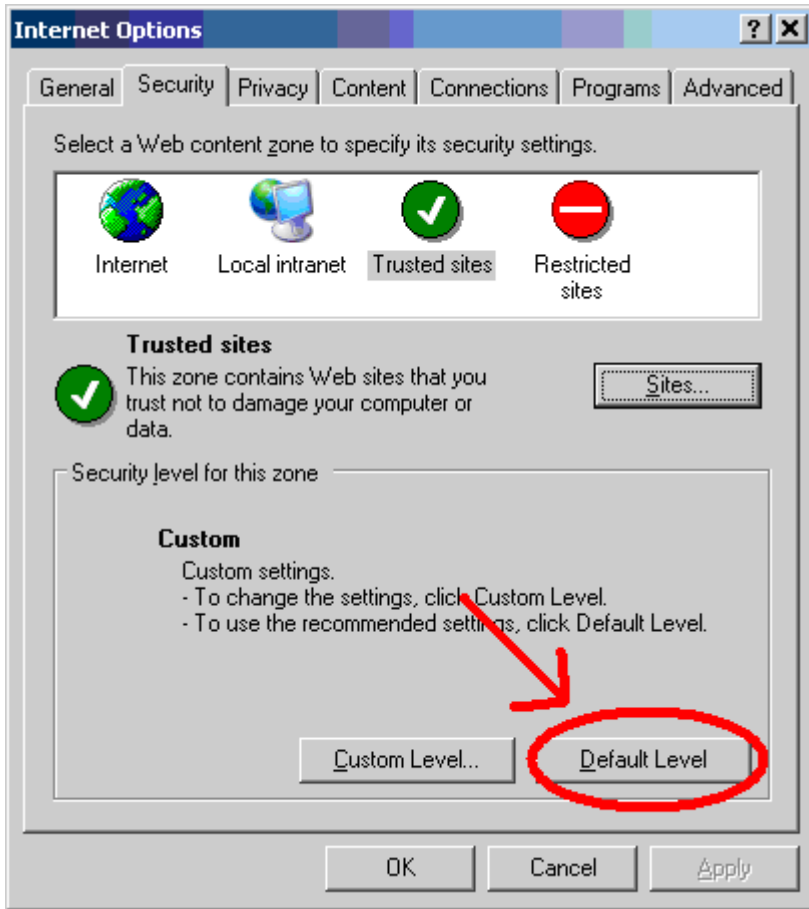


For Microsoft Internet 6 you can use the default setting of Low-Medium or Low by selecting “Default Level” and selecting “Low” or “Low-Medium”. Otherwise, select “Custom Level” and see the following for details.

For Microsoft Internet 7, you can use the default setting of Low by select “Default Level” and selecting “Low.” Otherwise, see next page for details.

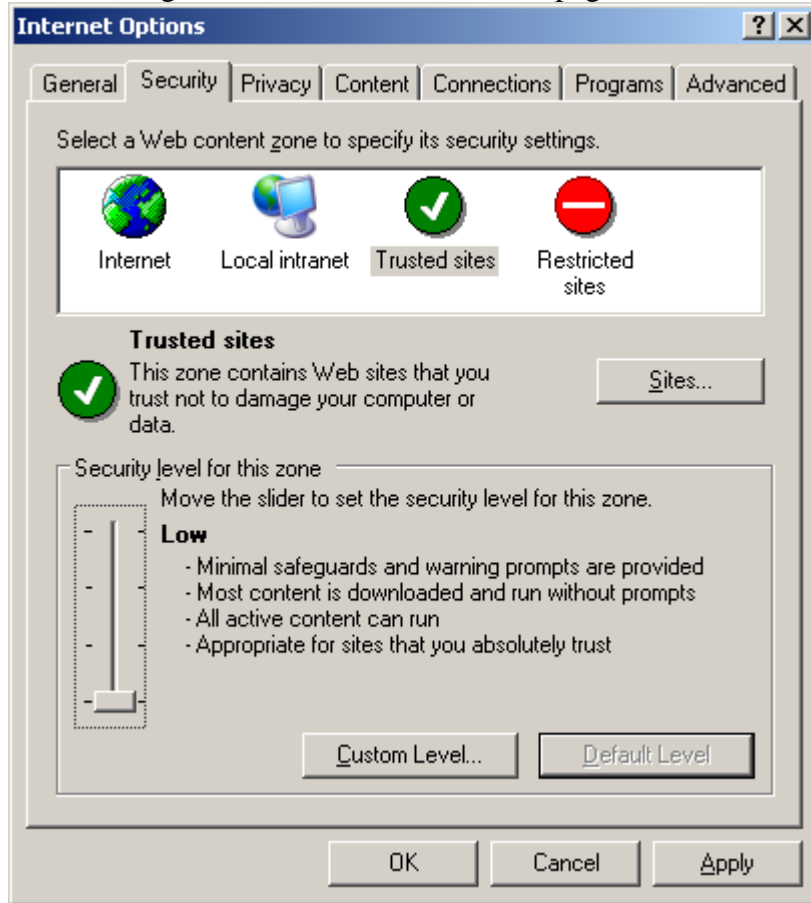

If you chose “Default Level”, skip to [3.\) Removing “reuse windows” for launching shortcuts.....](#) **Error! Bookmark not defined.**

Check the following **ONLY** if you are using **CUSTOM** security level.  
Make sure the “Trusted Sites” is highlighted and verify the following settings under “Custom Level”

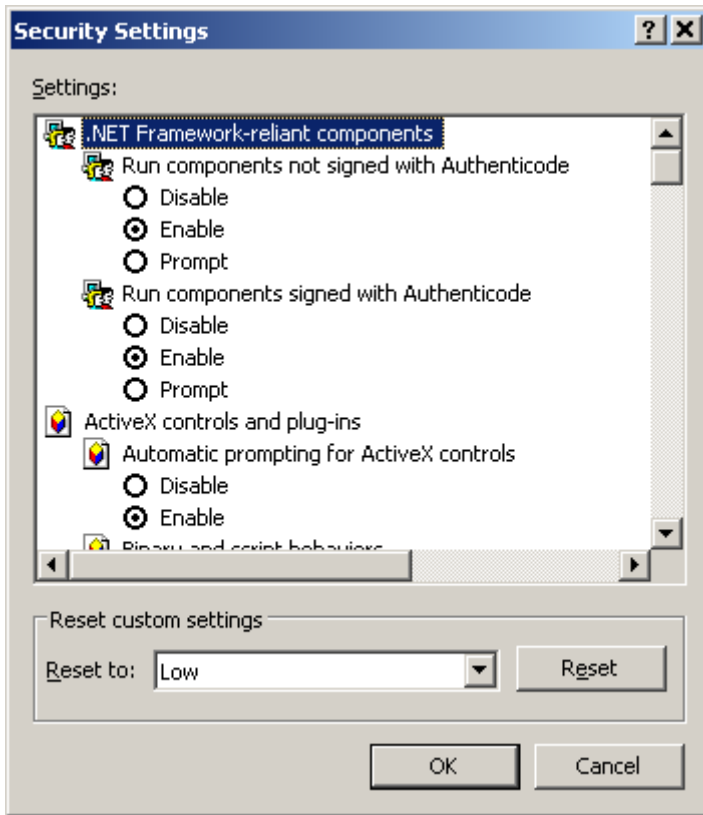

At the **Security Settings** window for the Trusted Sites zone, perform the following actions:

- Click on the **Enable** radio button for the **Automatic prompting for ActiveX controls** option (Windows XP, SP2 only).
- Click on the **Enable** radio button for the **Binary and script behaviors** option (Windows XP, SP2 only).
- Click on the **Enable** radio button for the **Download signed ActiveX controls** option.
- Click on the **Prompt** radio button for the **Download unsigned ActiveX controls** option.
- Click on the **Prompt** radio button for the **Initialize and script ActiveX controls not marked as safe** option.
- Click on the **Enable** radio button for the **Run ActiveX controls and plug-ins** option.
- Click on the **Enable** radio button for the **Script ActiveX controls marked safe for scripting** option.
- Click on the **OK** button at the bottom of the **Security Settings** window.


*The **Security Settings** window closes.*

### 3.) Removing “reuse windows” for launching shortcuts.

Click on the Advanced tab at the top of the Internet Options window, as shown below:


Verify the following settings.

- Scroll down the list of settings and, if selected, click on the **Reuse windows for launching shortcuts** checkbox to *deselect* it (this will prevent the application from launching in a previously opened Internet Explorer window), as shown below:

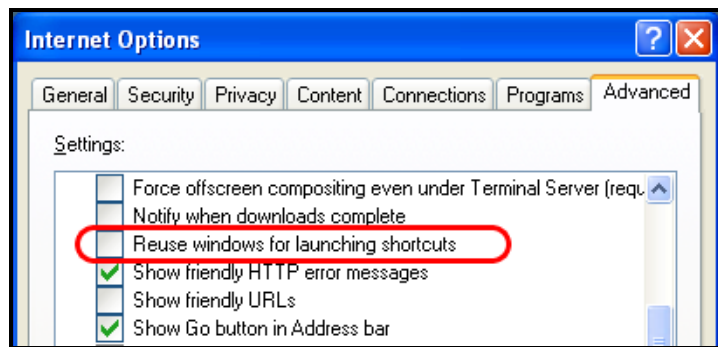

**Do Not Reuse Windows for Launching Shortcuts**

- If the client machine is running **Windows XP Service Pack 2**, scroll down the list of security settings and ensure that the **Allow active content to run in files on My Computer** checkbox is selected, as shown below:

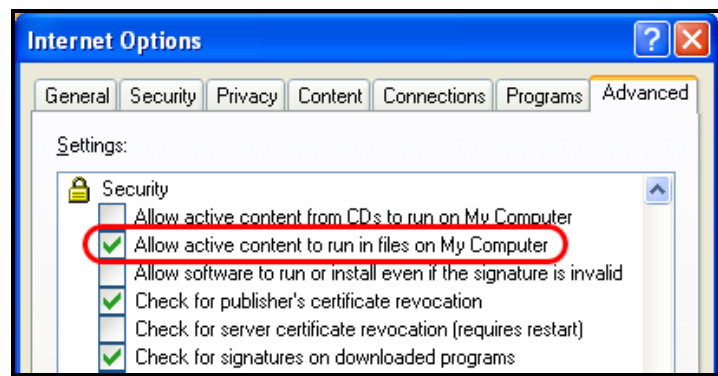

**Allow active content to run in files on My Computer**

1. Click on the **OK** button to exit the **Internet Options** window.

## 4.) Java Installation (Java 1.5.0.19)

The Amicas Viewer requires Java 1.5.#

[https://pacs.lincolnradiology.com/plugins/jre/1\\_5\\_0/](https://pacs.lincolnradiology.com/plugins/jre/1_5_0/)

**NOTE: Java 1.6.10 or 6.10 or higher breaks our viewer, please use the Control Panel, Add remove programs to remove this version(s).**

Here are two alternative links if you need version 1.5.19

[https://cds.sun.com/is-bin/INTERSHOP.enfinity/WFS/CDS-CDS\\_Developer-Site/en\\_US/-/USD/ViewProductDetail-Start?ProductRef=jre-1.5.0\\_19-oth-JPR@CDS-CDS\\_Developer](https://cds.sun.com/is-bin/INTERSHOP.enfinity/WFS/CDS-CDS_Developer-Site/en_US/-/USD/ViewProductDetail-Start?ProductRef=jre-1.5.0_19-oth-JPR@CDS-CDS_Developer)

or you can use this link


<http://java.sun.com/javase/downloads/5/jre>

**Install using typical defaults. Note Turn off automatic Updates via control panel – Java (Detailed instructions follows.)**

**Open the following link.**


[https://pacs.lincolnradiology.com/plugins/jre/1\\_5\\_0/](https://pacs.lincolnradiology.com/plugins/jre/1_5_0/)

It should show.

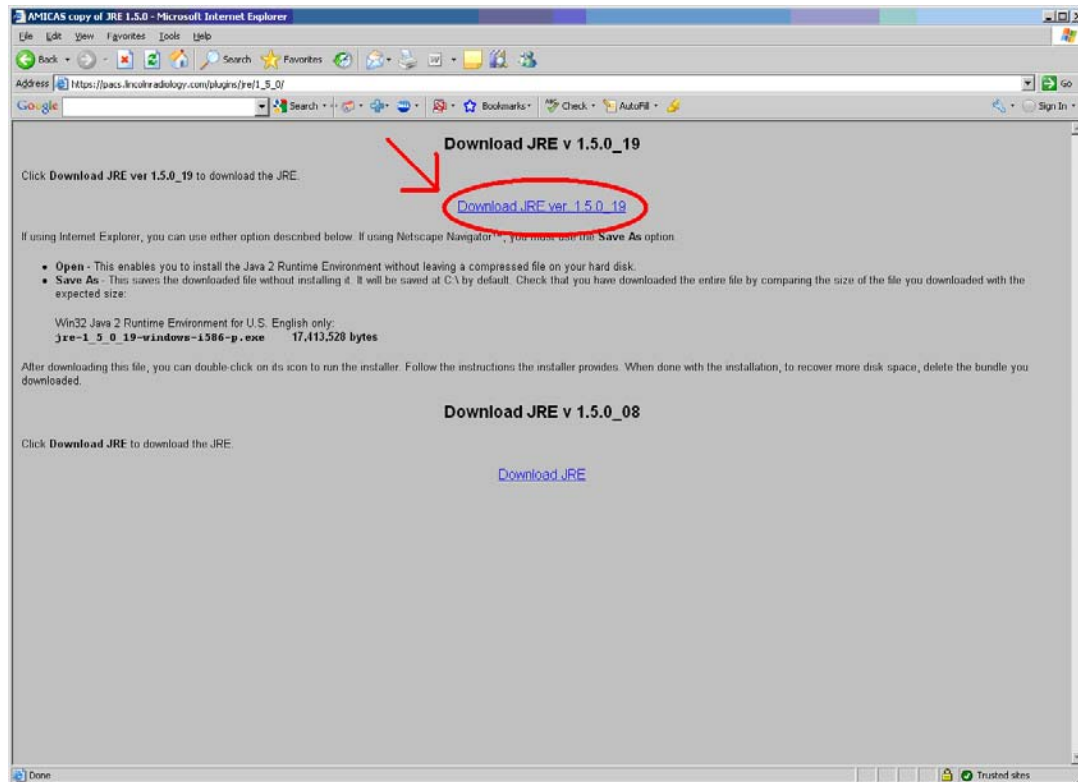



Select "Download JRE ver 1.5.0\_19.

Depending on your Internet Explorer settings you may get a File Download-Security Warning.

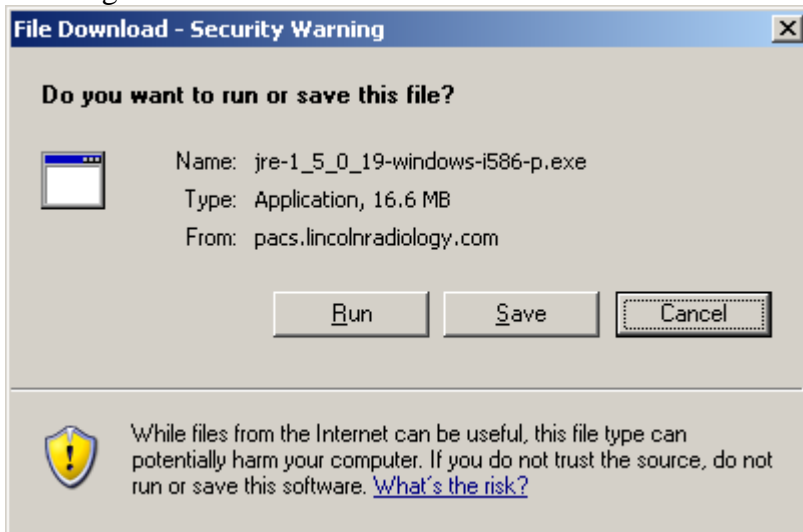

Select "Run".

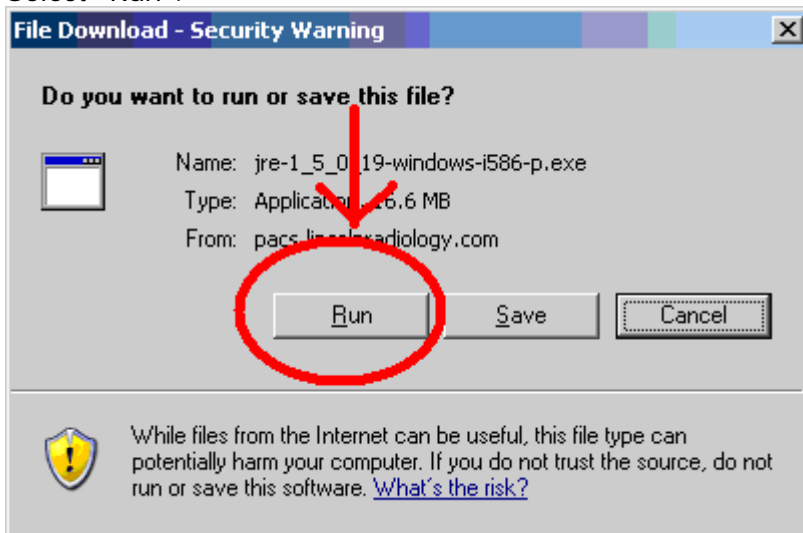

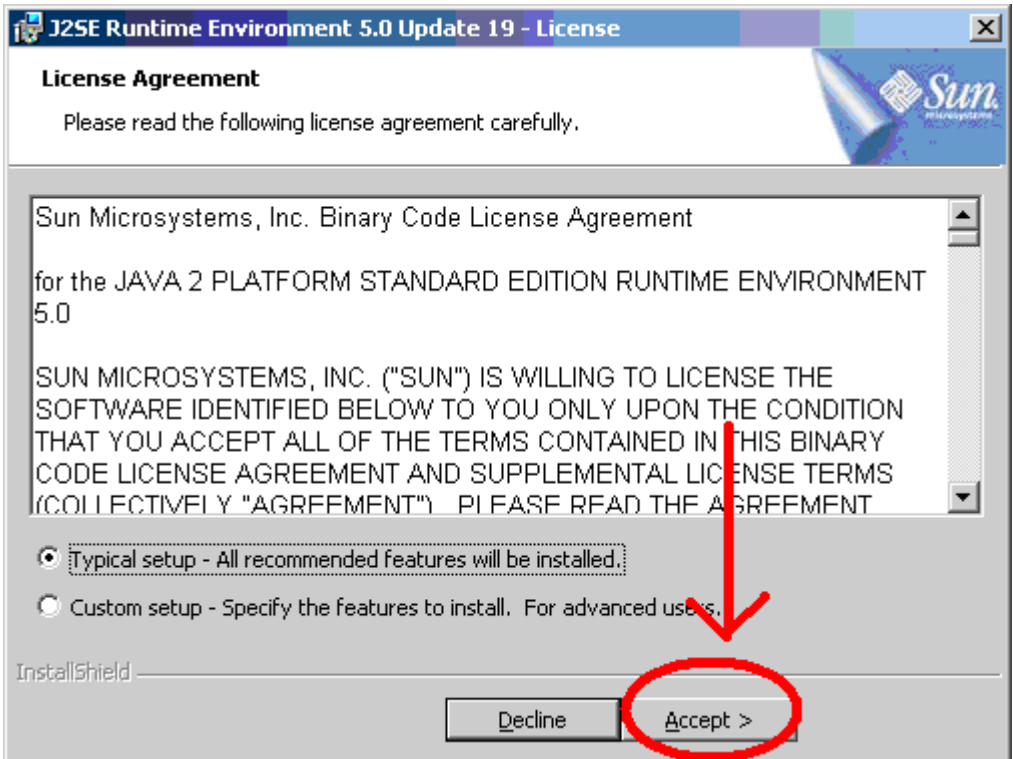


Once the license Agreement, shows select "Accept".



Once Installation Completed, select "Finish"

Now we need to turn off Java Automatic Updates.


Open Control Panel, (Under Start, Control Panel)

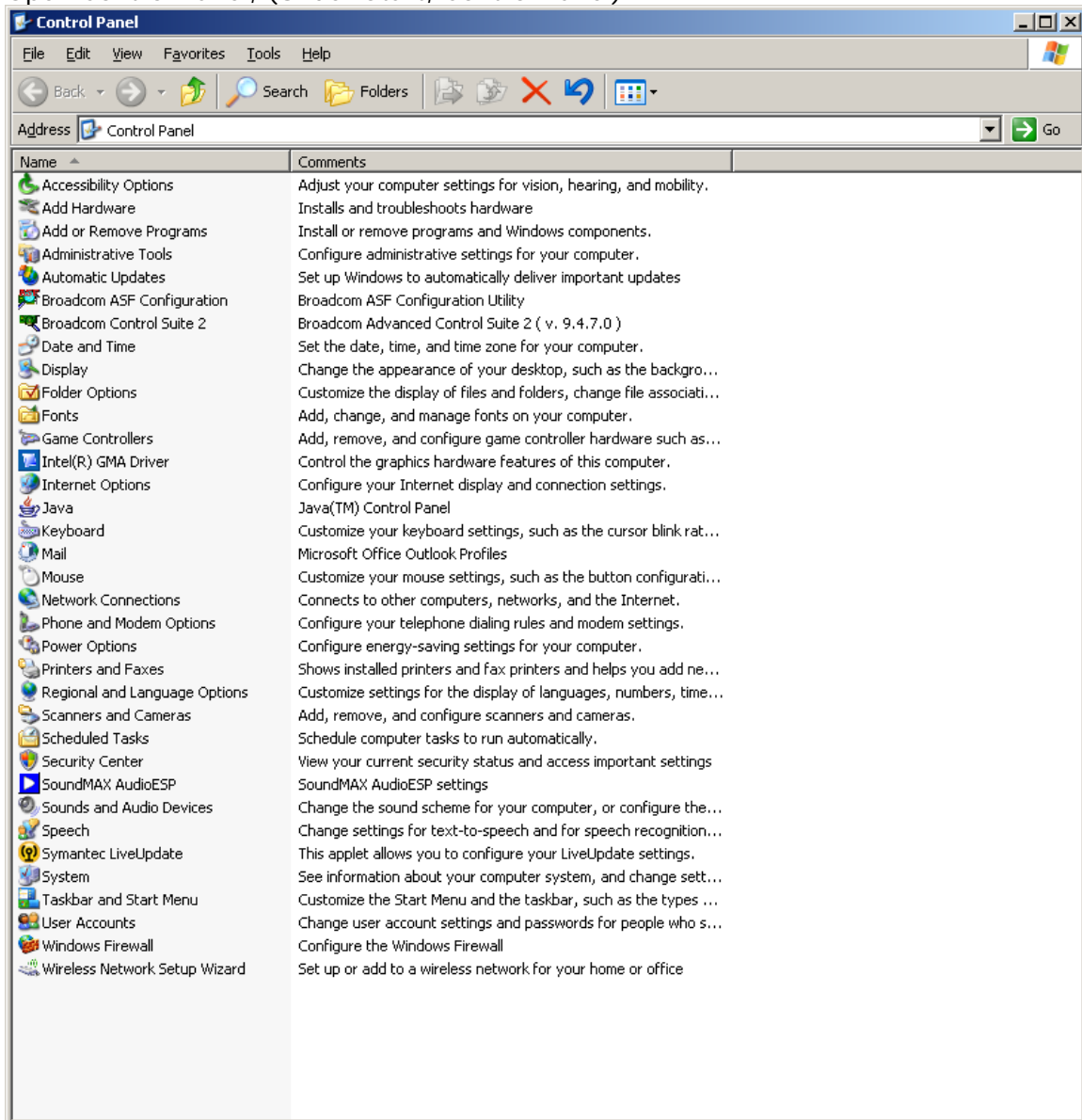


Select "Java", (The list should be in alphabetical order.)




Once the Java Control Panel opens, select the tab "Update".

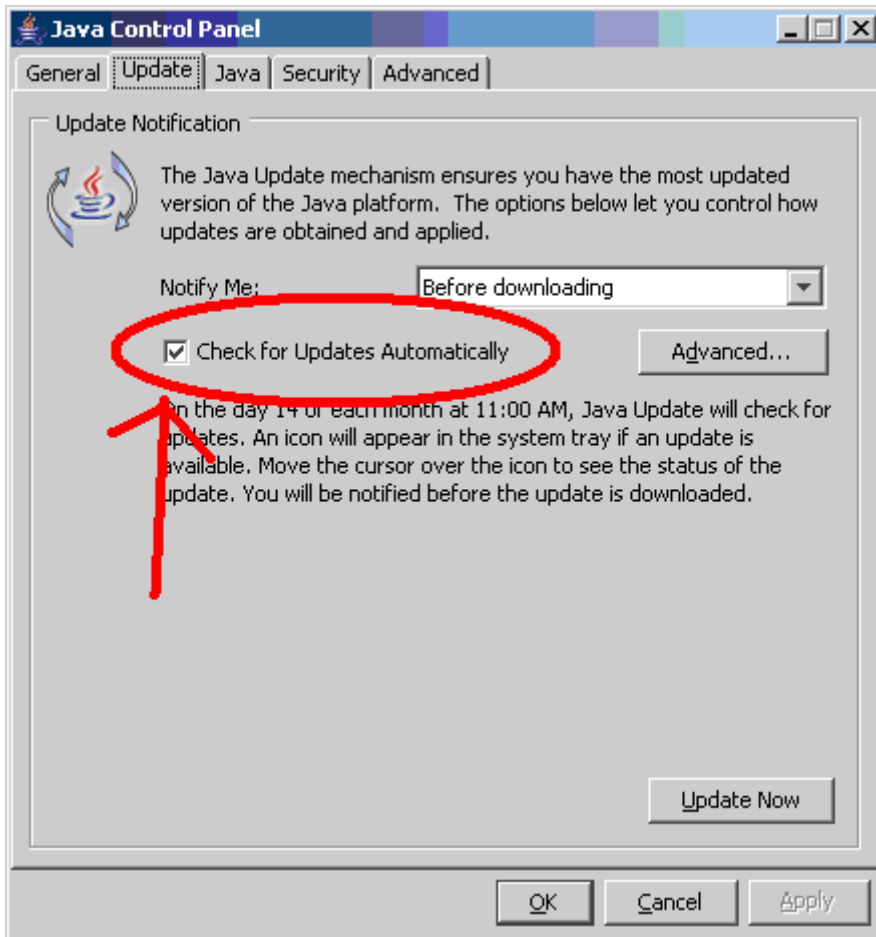

**Uncheck** the “Check for updates Automatically.”

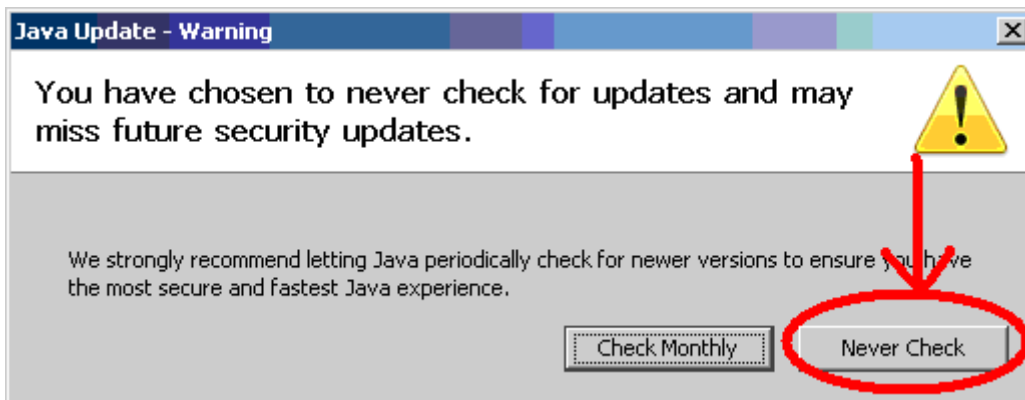

Select the “**Never Check**”

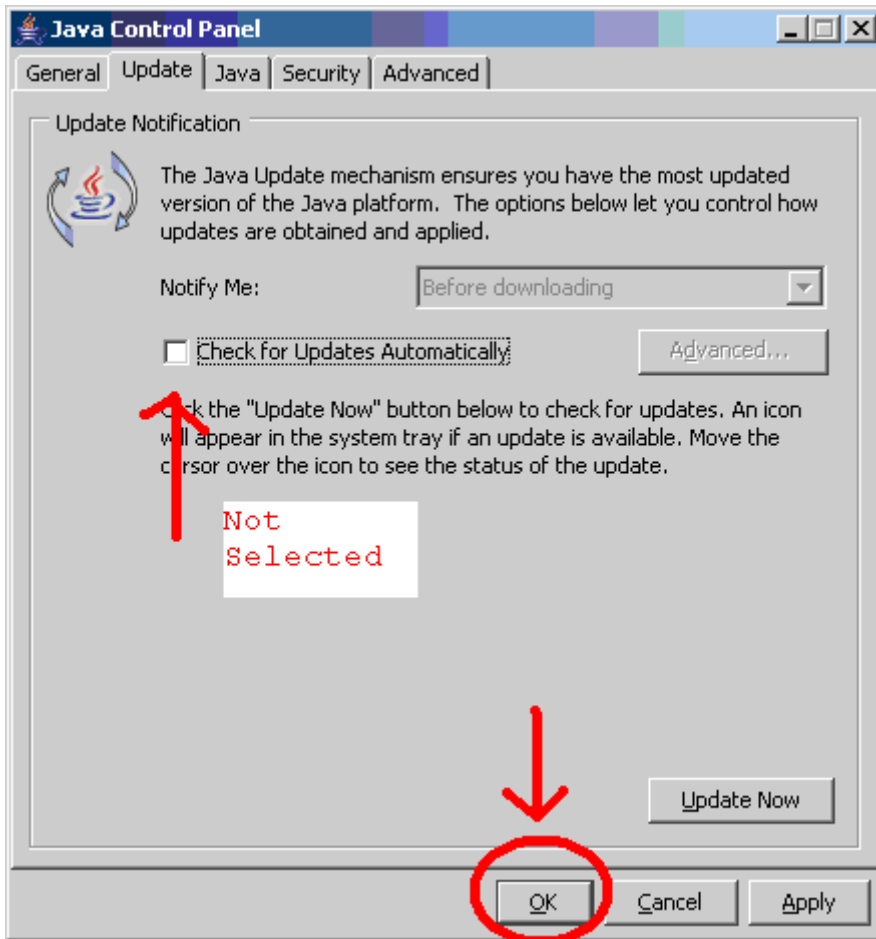


Notice that "Check for Updates Automatically" is NOT selected. Select "OK".

## 5.) Clear Internet Cookies, Files and History


On the menu, select Tools, Internet Options, Delete Cookies, Files, and History.

(Screen shots are from Internet Explorer 6.X)


It should show something similar to the following:

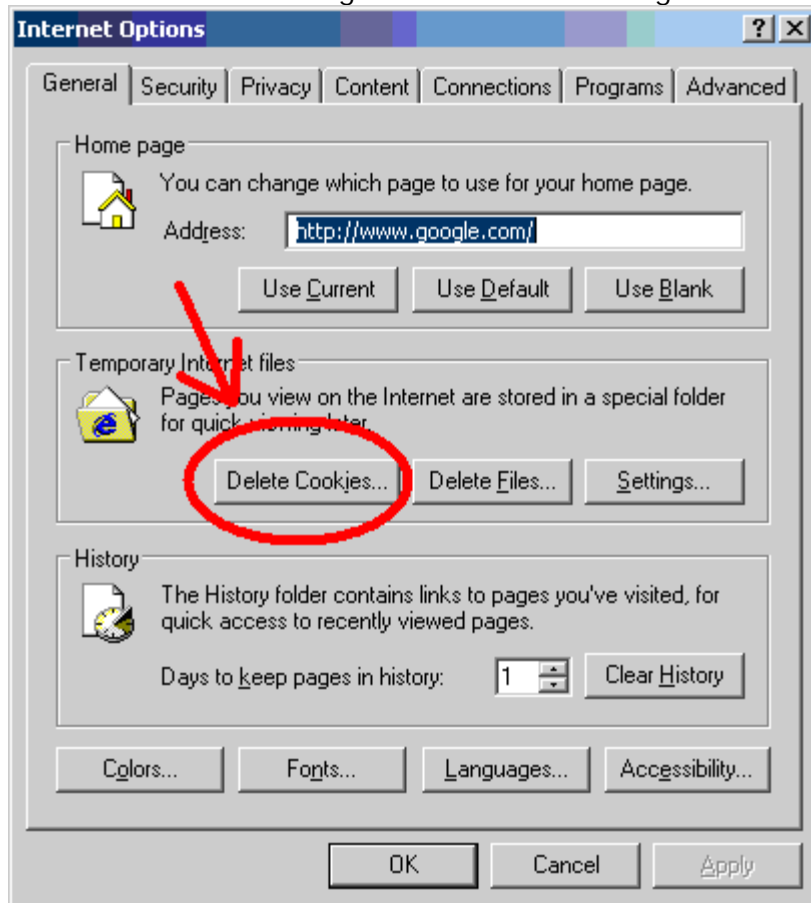


Select "Delete Cookies"

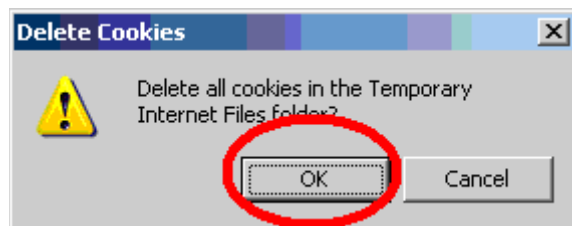


Select OK

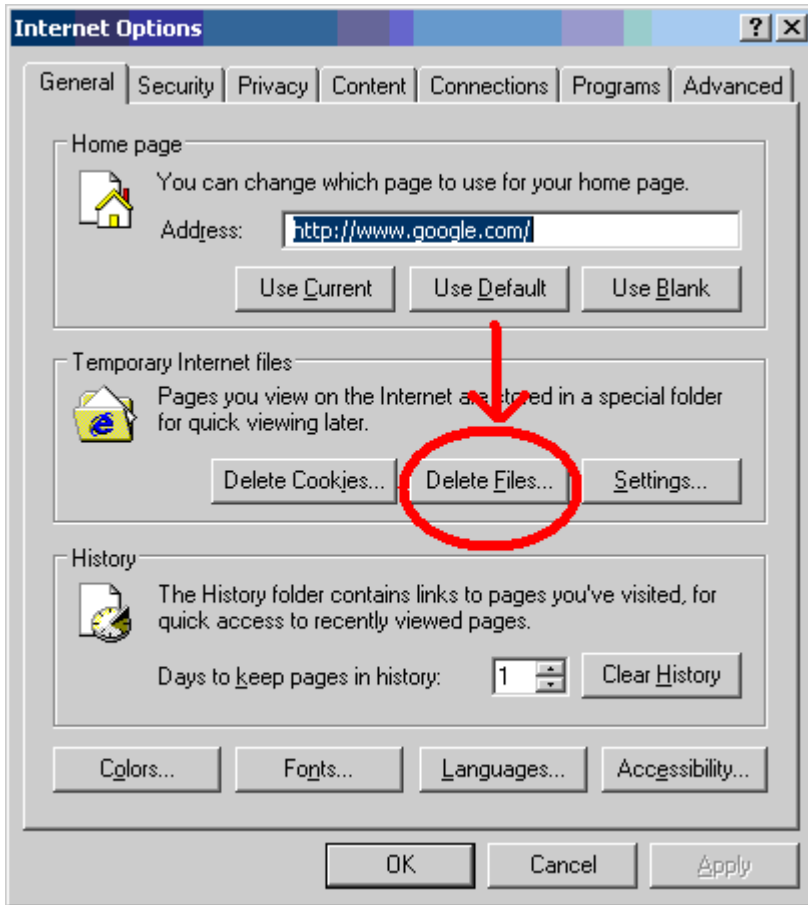


Select "Delete Files"

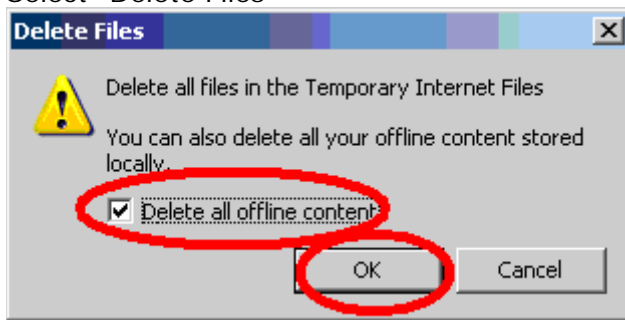

Select "Delete all offline content" and select "OK"

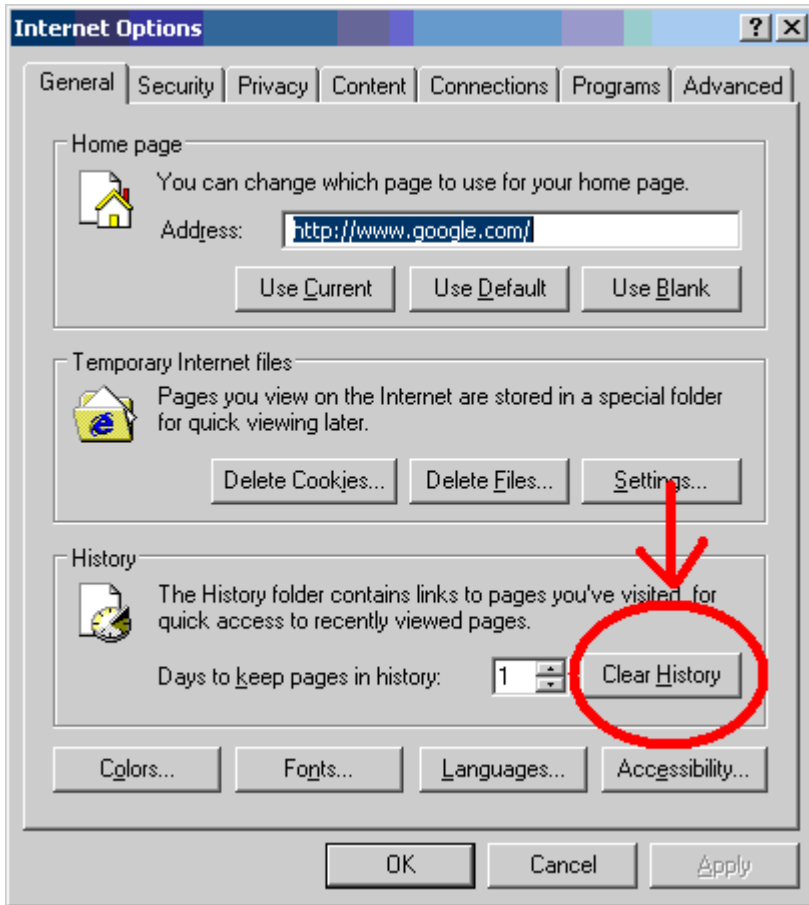

Select "Clear History"

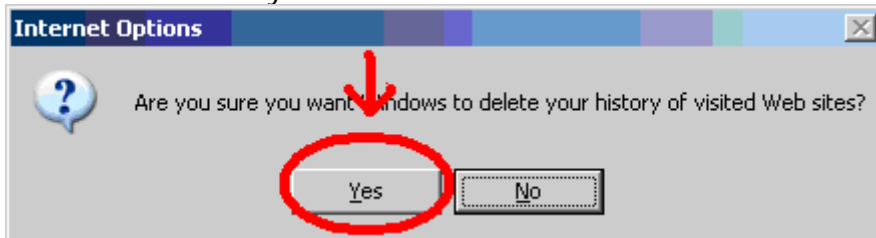


Select "YES"

## 5.) Adding a Shortcut

On the Desktop, (or where you want the short cut) in an open area, right click the desktop.


Select “New” – “Shortcut”

The following should appear...

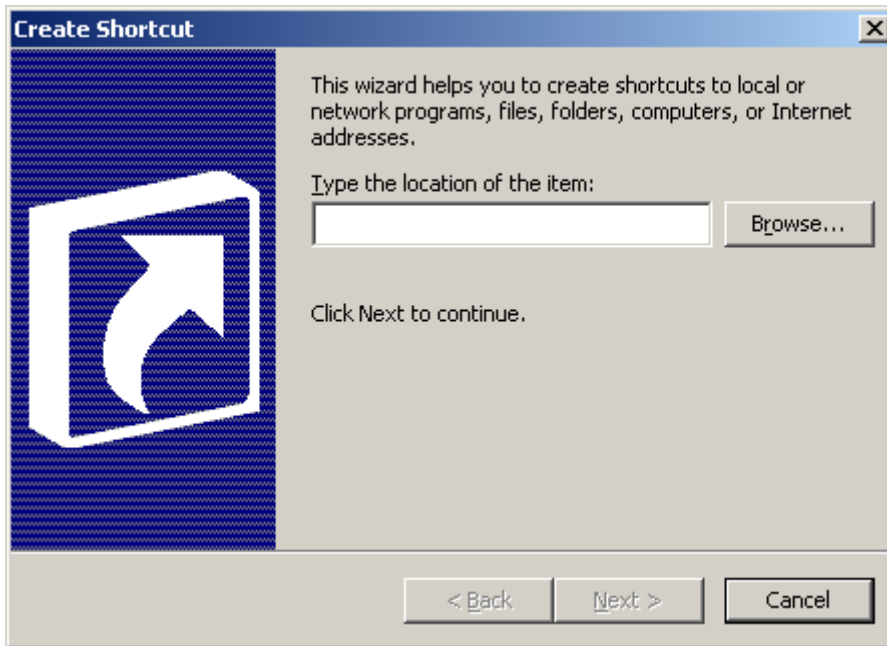


Either copy or type in the following:


<https://images.lincolnradiology.com/amicas-reach/>

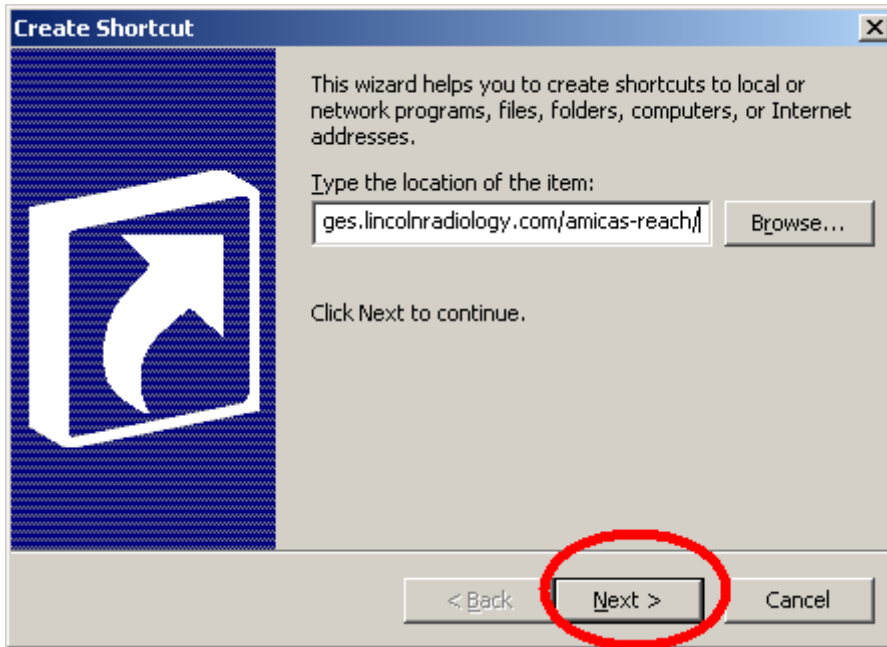

After typing, <https://images.lincolnradiology.com/amicas-reach/>  
Select “Next”

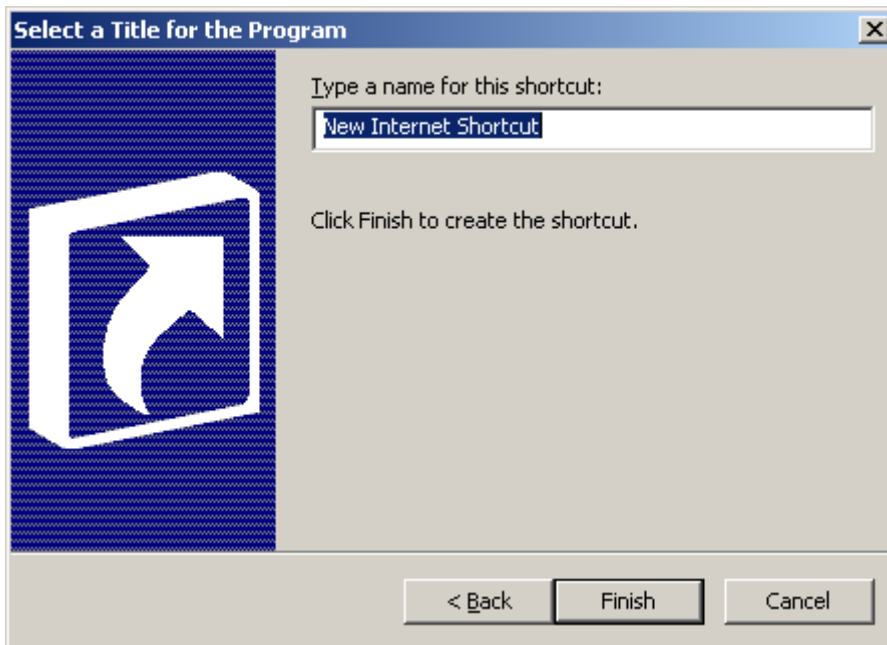

Type in “Lincoln Radiology” for the name.

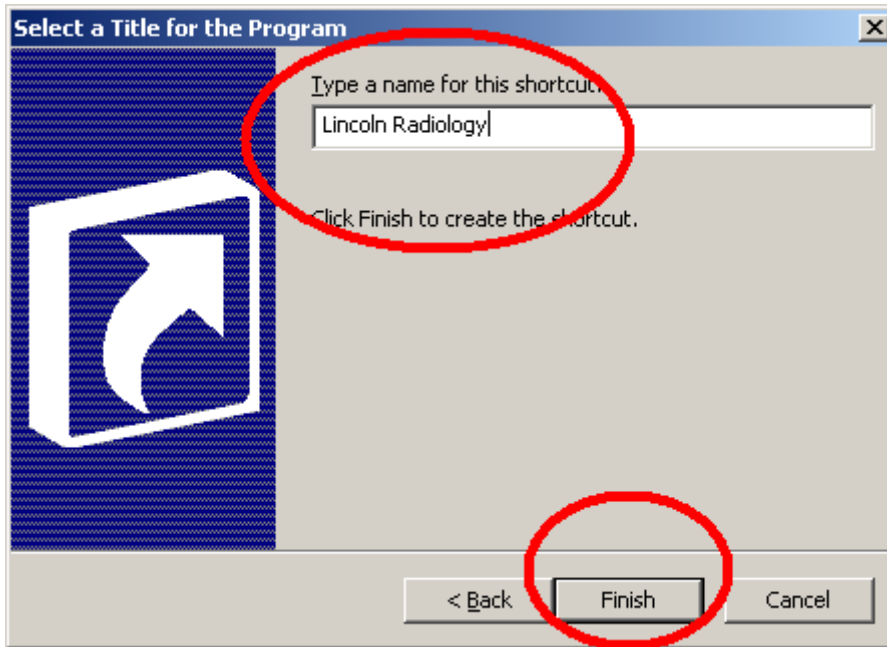

When done typing in "Lincoln Radiology" for the name, select "Finish"

## Miscellaneous Notes:


1.) When opening an image and just black screen delete Amicas Folder found in C:\Program Files directory and reopen. Reinstalls the product...

### 2.) Vista

(Make sure to run first time Internet Explorer as Administrator)


Start, Programs, right click on Internet Explorer, run as Administrator

3.) Java Security Warning – download Java 1.5\_19

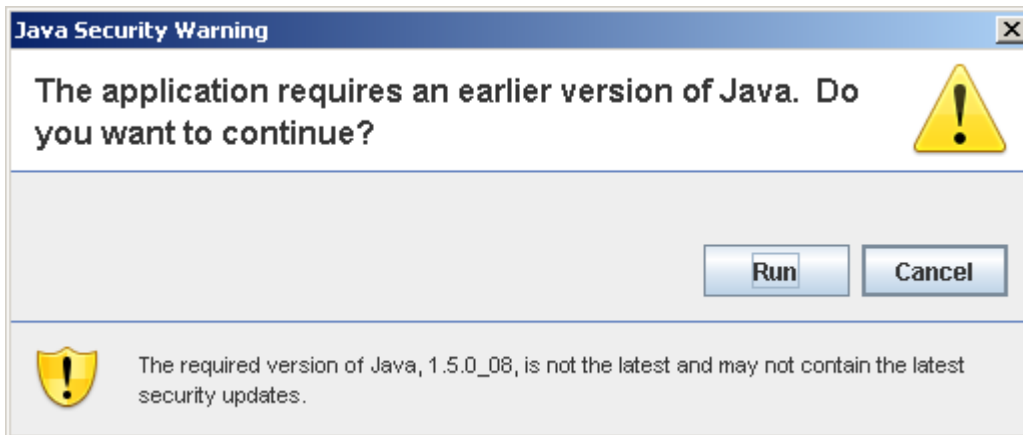

If you see the above Java Security Warning, Install Java Runtime Environment 5.0 version 19. You may need to uninstall any Java Version 6.10 or higher.

4.) See grayed out screen with icons grayed out.

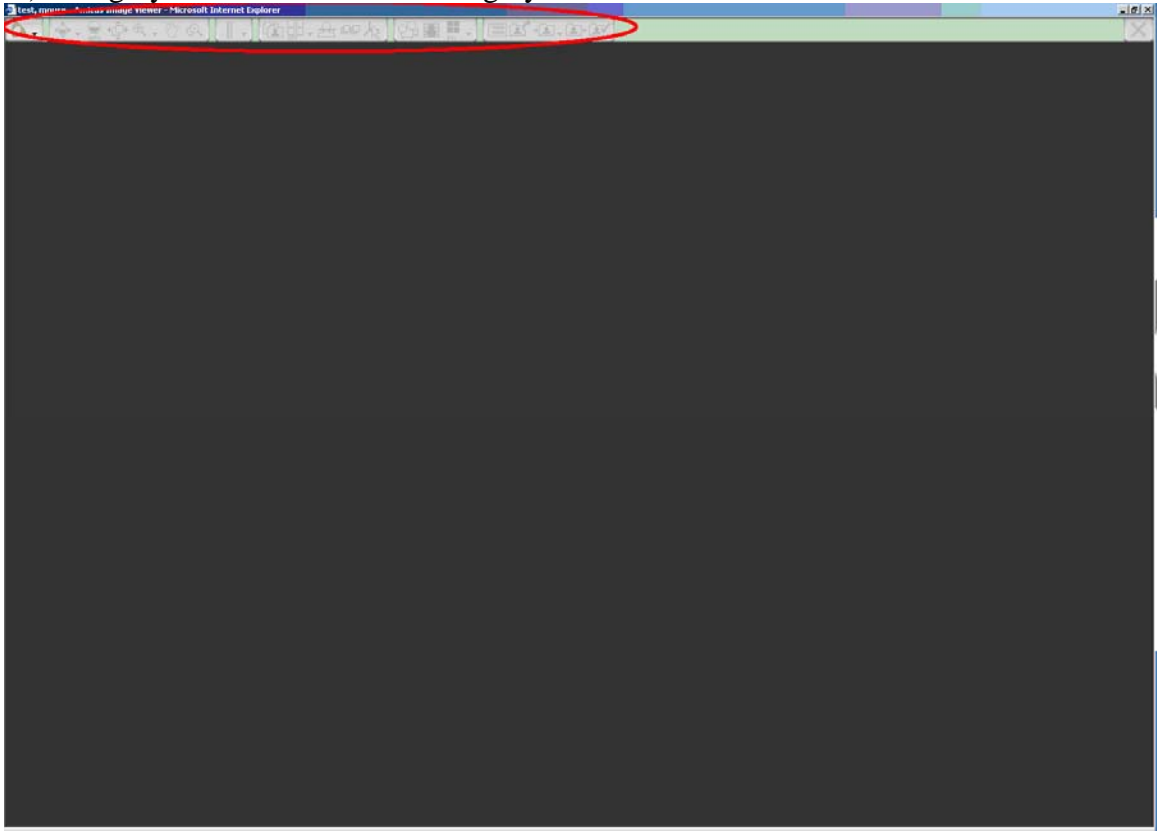

Make sure Java 6\_10 or higher is uninstalled, by using Control Panel, Add Remove Programs. (e.g. 6\_10, 6\_11, 6\_12, 6\_13, 6\_14,...)